

Instructions for use

FOR STERILE COVERAGE OF THE ULTRASONIC HEAD

GEL-FREE
FLEXASOFT®
ultrasonic - cover

1. Carefully remove ultrasonic cover from its sterile packaging. Special attention must be paid to properly unfolding the cover, otherwise cover will be rendered unusable. Unfold the cover so that the white rectangular section is facing up.
2. Starting at the end of the cover marked by the yellow tab, the **sterile person** should now reach into the cover FOLLOWING THE PATH OF THE LONG THIN WHITE TAB (a). (Along direction of arrows.) Keep this hand in this position.
3. The sterile person - with his/her free hand - should now remove the white rectangle to expose the adhesive/gel area.
4. The NON-STERILE person now places the ultrasound head onto the adhesive/gel area. To prevent wrinkling of cover, STERILE person should rub/flatten the adhesive/gel area against the ultrasound head(with hand that is inserted inside cover.)
5. With the STERILE person firmly holding the ultrasound head, the NON STERILE person grabs the yellow tab (b) and pulls cover over head and cable.
6. The STERILE PERSON should now remove their hand from the inside cover WHILST TAKING THE THIN WHITE STRIP with them.
7. Remove adhesive from thin white strip and use to wrap around probe cover.

